

PREPARATION



1. Clean the probe with a validated product. (***)

DISINFECTION




2. Dispense two aliquots of Tristel Duo onto a dry wipe (Duo Wipe is recommended).



3. Use the wipe to spread the foam over the surface of the probe to cover all areas. (**)




4. Leave the surface to dry to ensure a minimum 30 sec contact time.

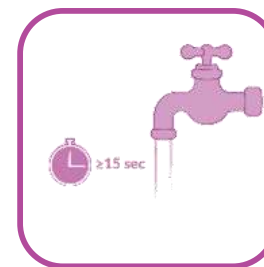

5. Rinse the probe thoroughly with deionized water or use Tristel Rinse Wipe to remove excess foam.

DRYING

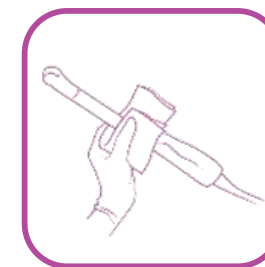

6. Carefully dry the probe with a sterile soft, dry cloth or gauze.