

Tristel Duo™ ULT



Tristel Duo™ ULT is a high-level disinfectant foam designed for ultrasound probes.

QUICK GUIDE
Tristel Duo™
ULT

Tristel Duo™ ULT

Validated disinfection product for Esaote's probes*

QUICK GUIDE
Tristel Duo™
ULT

CLEANING




1. Clean the probe with a validated product. (**)

DISINFECTION




2. Dispense two aliquots of Tristel Duo™ ULT onto a dry wipe (Duo Wipe is recommended).

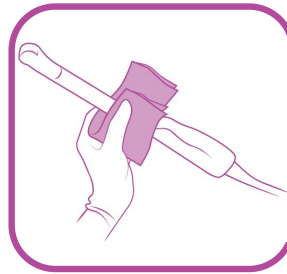

3. Use the wipe to spread the foam over the surface of the probe to cover all areas. (***)




4. Leave the surface to dry to ensure a minimum 30 sec contact time.

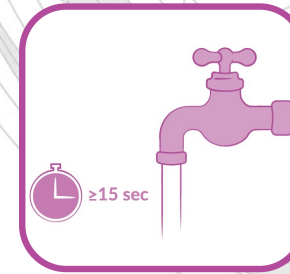

5. Rinse the probe thoroughly with deionized water or use Tristel™ Rinse Wipe to remove excessive foam.

DRYING

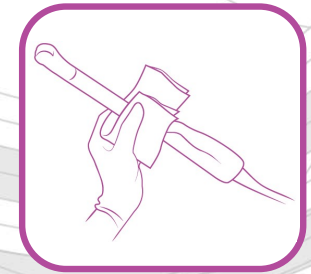

6. Carefully dry the probe with a sterile soft, dry cloth or gauze.

* For Esaote probes list validated with Tristel Duo™ ULT, please refer to the *MyLab™ Probes and Consumables manual*.

** Use products approved by the Esaote (ref. to *MyLab™ Probes and Consumables manual*) following the product manufacturer's instructions.

*** After the usage: discard the used wipe and gloves in accordance with local regulations. Do not re-use.