

# Tristel ULT™\*

## QUICK GUIDE Tristel ULT™



Tristel ULT™ is a high-level disinfectant foam designed for ultrasound probes. The package contains foam and wipes.

# Tristel ULT™\*

Validated disinfection product for Esaote's probes\*\*

QUICK GUIDE

Tristel  
ULT™

## CLEANING




**1.** Clean the probe with a validated product. (\*\*)

## DISINFECTION




**2.** Dispense four aliquots of Tristel ULT™ foam onto a Tristel ULT™ wipe. (\*\*\*\*) Gently close hand around the wipe and wait for 10 seconds. Do not squeeze.

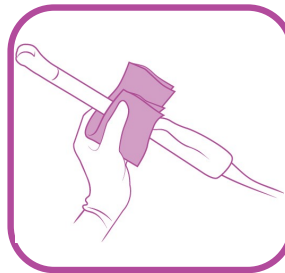

**3.** Wipe the probe to spread the foam using a massaging motion from one end of the probe to the other 4 times. Cover the entire surface of the probe including the shaft and handle. (\*\*\*\*\*)




**4.** After wiping, leave the probe undisturbed. Place the probe on a clean surface to avoid recontamination. The contact time for high-level disinfection is 2 minutes.

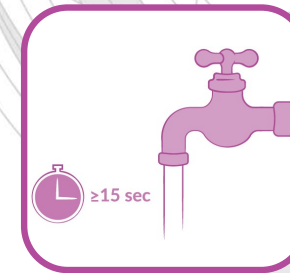

**5.** Rinse the probe thoroughly with deionized water to remove excess foam. (\*\*\*\*)

## DRYING

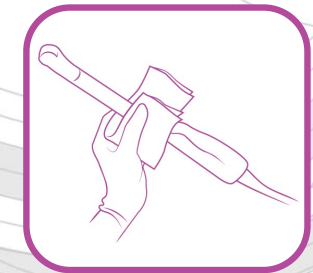

**6.** Carefully dry the probe with a sterile soft, dry cloth or gauze.

\* FDA-cleared

\*\* For Esaote probes list validated with Tristel ULT™, please refer to the *MyLab™ Probes and Consumables manual*.

\*\*\* Use products approved by the Esaote (ref. to *MyLab™ Probes and Consumables manual*) following the product manufacturer's instructions.

\*\*\*\* In USA Tristel ULT™ must be used with Tristel ULT™ Wipes.

\*\*\*\*\* After the usage: discard the used wipe and gloves in accordance with local regulations. Do not re-use.

\*\*\*\*\* Tristel™ Rinse Wipes are not available in the USA.

Tristel™

Esaote S.p.A. - sole-shareholder company - Via Enrico Melen 77, 16152 Genova, ITALY, Tel. +39 010 6547 1, Fax +39 010 6547 275, [info@esaote.com](mailto:info@esaote.com)

Tristel ULT is a trademark of Tristel plc.

MyLab is a trademark of Esaote spa. Technology and features are device/configuration dependent. Specifications subject to change without notice. Information might refer to products or modalities not yet approved in all countries. Product images are for illustrative purposes only. For further details, please contact your Esaote sales representative.

esaote  
HEALTH WITH CARE