

# Tristel Trio™

Validated disinfection product for Esaote's probes\*

QUICK GUIDE  
**Tristel Trio™**  
Wipes System



Tristel Trio™ is a three-step reprocessing system designed for the high-level disinfection of ultrasound probes.

The box contains:


- Tristel™ Pre-Clean Wipe
- Tristel™ Sporicidal Wipe with the Activator Foam
- Tristel™ Rinse Wipe

# Tristel Trio™


Validated disinfection product for Esaote's probes\*

QUICK GUIDE  
**Tristel Trio™**  
Wipes System


## PRE-CLEANING

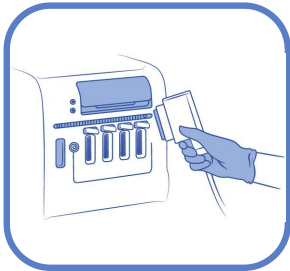

**1.** Disconnect the probe from the system.

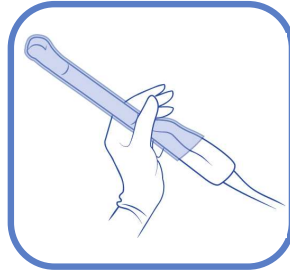


**2.** Remove the protective cover, and disassemble any accessories, if any.

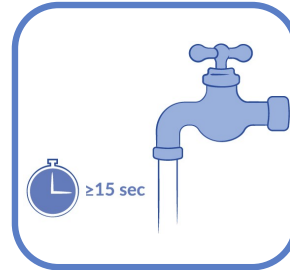


**3.** Wash off all residuals at T<60°C for at least 15 sec and no more than 15 min after the use.

## CLEANING

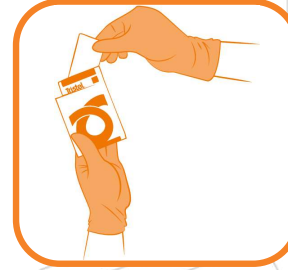


**4.** Disinfect hands and wear gloves.

**5.** Take one Pre-Clean Wipe sachet and remove the Wipe from its sachet.

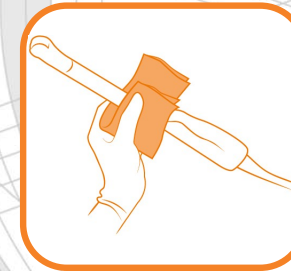

**6.** Wipe the surface of the probe until visually clean. (\*\*)

## ACTIVATING & HIGH-LEVEL DISINFECTING

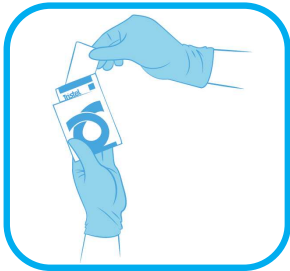

**7.** Disinfect your hands and wear new gloves.


**8.** Take one Sporicidal Wipe sachet and remove the Wipe from its sachet.



**9.** Remove the lid from the Activator Foam bottle. Apply two aliquots of Activator Foam onto the Wipe.




**10.** Fold the Wipe in on itself and scrunch together for 15 sec to activate and use it immediately.

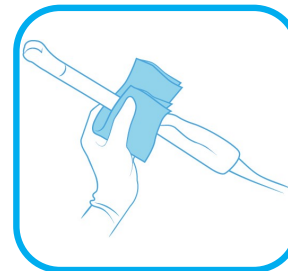


**11.** Wipe the surface of the probe in one movement to cover all areas with the foam.

**12.** Observe a 30 sec contact time. (\*\*)

## RINSING

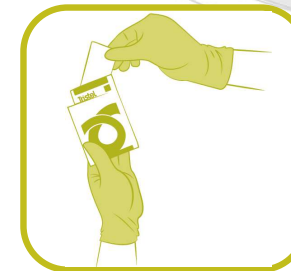

**13.** Take one Rinse Wipe sachet and remove the Wipe from its sachet.

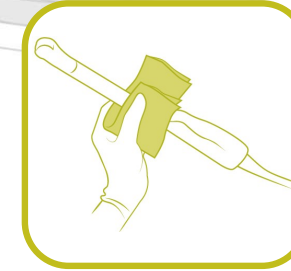

**14.** Wipe the surface of the probe to remove excess foam. (\*\*)

**15.** The probe should be left to air dry.

\* For Esaote probes list validated with Tristel Trio™, please refer to the *MyLab™ Probes and Consumables manual*.

\*\* After the usage: discard the used wipe and gloves in accordance with local regulations. Do not re-use.

**Tristel™**

Esaote S.p.A. - sole-shareholder company - Via Enrico Melen 77, 16152 Genova, ITALY, Tel. +39 010 6547 1, Fax +39 010 6547 275, [info@esaote.com](mailto:info@esaote.com)

Tristel Trio is a trademark of Tristel plc.

MyLab is a trademark of Esaote spa. Technology and features are device/configuration-dependent. Specifications subject to change without notice. Information might refer to products or modalities not yet approved in all countries. Product images are for illustrative purposes only. For further details, please contact your Esaote sales representative.

**esaote**  
HEALTH WITH CARE